

## CC-50 AED Alarm Cabinet

### ALARMED CABINET WITH ARC APPROVED SIGNAGE

This alarmed cabinet deters would be thieves as well as announcing an emergency as soon as the door is opened. The battery operated alarm will sound once the door is opened, and a strobe light will flash.

The AED is visible through the perspex cabinet front.

**Code:** CC-50

**Size:** HS 38 W42 D15.5 cm

**Colour:** White

**Warranty:** 2 years




### INSTALLATION GUIDE AND OPERATING INSTRUCTIONS

1. Hold the wall cabinet at a suitable height
2. Mark on the wall the positions of the 2 fixing holes, visible from the inside of the cabinet
3. Secure the cabinet using appropriate fixing methods through these holes
4. Place operating AED in alarm cabinet
5. Switch on alarm and check operation by opening the door and closing it again.  
The cabinet installation is now complete

### TROUBLE SHOOTING

Battery replacement should only be necessary after several years. A simple monthly check is to briefly open and close the cabinet door to ensure that the alarm and light are functioning properly. When the battery is low the alarm might function without the light. To replace the battery, see below instructions

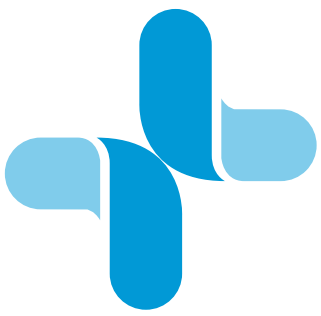

# CC-50 AED Alarm Cabinet

## Instructions for battery replacement

### 1. Locate the battery box



### 2. Slide out the battery compartment



### 3. Slide off battery cover



### 4. Replace Battery

